Estimado paciente!

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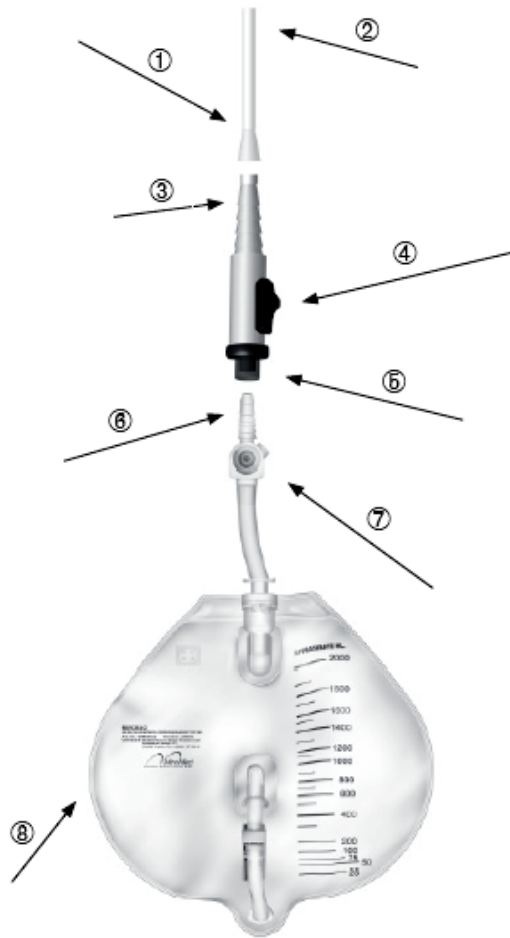
Dear patient!

To make the usage of a foley catheter easier, we have developed this modern catheter valve. Please read the operating instruction attentively before using the valve.

Your MiroMed® team wishes you a fast recovery

unpacking

To make the usage of a foley catheter easier, we have developed this modern catheter valve. Please read the operating instruction attentively before using the valve.

Attachment to the catheter

Hold the valve (4) so that you can see the switch. The conical attachment (3) points to you. Strongly press the conical attachment (3) into the funnel attachment (1) at the catheter. The valve is designed to independently stick to the catheter.

Emptying the bladder

In order to empty the bladder you hold the front end (5) of the valve above the toilet washbasin or above another receptacle and pull the slidegate valve (4) in the direction of the catheter. After emptying your bladder, let the slidegate valve go (4); the valve closes automatically.

Nightly use

At night you put the conical attachment of the urine bag (6) into the front end (5) of the valve. The special elastic material causes a self bonding. In this position the valve is open and the urine can drain off. Take care that the drain tube (7) is not snapped off. In the morning hold the valve tight at the front end (5) and unplug the connection of the valve (6) to the front valve end (5). The valve closes automatically. Now you can remove the urine bag (8).

Cleaning of the valve

Hold the valve tight at the front end (5) and unplug the connection of the valve (6) to the front valve end (5). The valve closes automatically. Spray the front valve end (5) with a medical disinfectant and let the means have an effect. Remove the conical attachment of the valve (3) from the funnel attachment as seldom as possible to reduce the risk of a germ colonisation.

Liability disclaimer

In the case of preparation for re-use, the physical, chemical and / or biological properties of the product may be altered.

* For further questions, please consult your attending doctor.

Watch the video !

Copyright and source for purchase:
MiroMed Pfeleiderer GmbH
 Lindleystr.12, 60314 Frankfurt am Main
 Telephone: +49 69 48 98 78 0
 Telefax: +49 69 48 98 78 70
 Web: www.miroalve.de
 Email: mail@miromed.eu

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MiroMed Pflleiderer GmbH